

## Durchführung der Probenentnahme für den PCR-Test

1. Öffnen Sie die Salzlösung und nehmen Sie diese in den Mund.
2. Spülen Sie **bei geschlossenem Mund 1 Minute lang** gründlich (ähnlich wie Mundwasser).
3. Über dem Waschbecken transferieren Sie nun die Salzlösung in das Probenröhrchen.
4. Verschließen Sie das Probenröhrchen.
5. Stellen Sie das verschlossene Probenröhrchen in den dafür vorgesehen Ständer.
6. Bewahren Sie den Zettel mit Ihrem QR-Code auf, um später das Test-Zertifikat abrufen zu können ([www.testung.at/ergebnis](http://www.testung.at/ergebnis)).
7. Desinfizieren Sie Ihre Hände!

## Carrying out sample collection for the PCR-test

1. Open the saline solution and put it in your mouth.
2. Rinse thoroughly for **1 minute with your mouth closed**.
3. Above the sink transfer the saline solution into the sample tube.
4. Close the sample tube.
5. Place the sealed sample tube into the racks provided.
6. Keep the paper with the QR code to later retrieve your test certificate ([www.testung.at/ergebnis](http://www.testung.at/ergebnis)).
7. Disinfect your hands!